

Smart Hearing

App Version 1.0


User Guide

*Android phones | iPhone | iPad | iPod



JINGHAO

Introduction

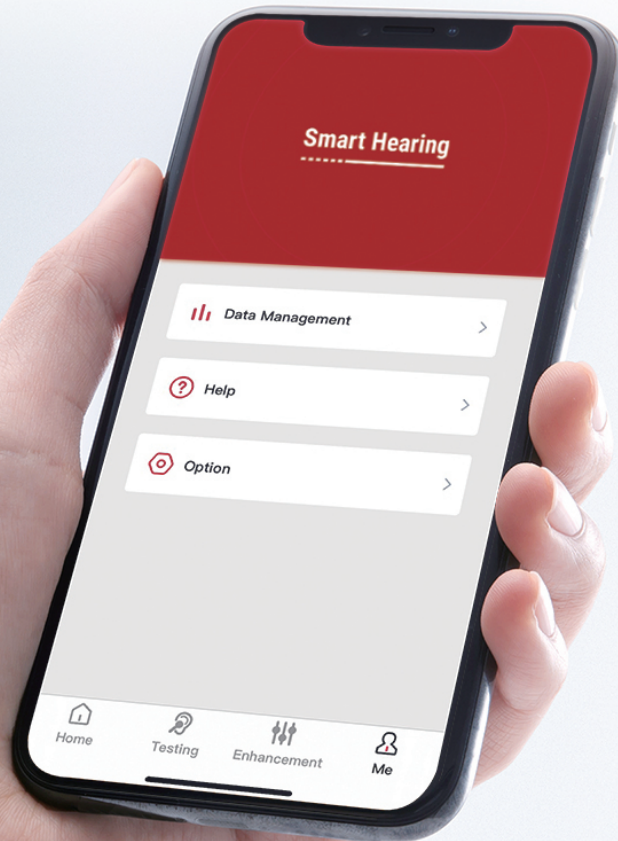

Read all the information in this instructions before using.

Smart Hearing are made to enhance your hearing experience.

For more information and help, please visit website:
www.jinghao.cc

Printed version is available via customer support.

Contents

QUICK OVERVIEW

DOWNLOAD THE “SMART HEARING”

PAIRING WITH YOUR HEARING AID

HOME SCREEN FUNCTIONS

**iPhone® is a trademark of Apple Inc., registered in the U.S. and other countries.*

Android™ is a trademark of Google Inc.

ios is a trademark of Cisco Inc.



Smart Hearing

【DOWNLOAD THE SMART HEARING APP】

*The "Smart Hearing" is compatible with Android and iOS.

1.1 App Store


Download the "Smart Hearing" for Apple devices

The "Smart Hearing" makes it easy to control and adjust your hearing aids with your compatible Apple device. Access apps for iPhone, iPad and iPod touch - download on the App Store by following these steps:

Step One: On a compatible Apple device, click on "App Store."



Step Two: Type in "Smart Hearing" in the search box.

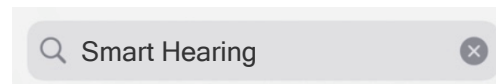

Step Three: Tap "GET" on your device. Follow the screen prompts to finish downloading.

GET

Step Four: Once the app has installed, tap the app from your Home screen or Apps Library.



1.2 Goodle Play Store

Download the "Smart Hearing" for Android devices

The "Smart Hearing" makes it easy to control and adjust your hearing aids with your compatible Android device. Downloading the app by following these steps:

Step One: On a compatible Android device, click on "Google Play Store."



Step Two: Type in "Smart Tune" in the search box.

← Smart Hearing 🔍

Step Three: Tap "GET" to download. Follow the screen prompts to finish downloading.

GET

Step Four: Once the app has installed, tap the app from your Home screen or Apps Screen.



【PAIRING WITH YOUR HEARING AID】

1. Turn on the hearing aids.
2. Enable Bluetooth on the phone.
3. Search the name “G.SoundBuds” of the device.
4. Tap to connect your hearing aid with your phone.
5. Tap "Smart Hearing" to started. On the botton of first screen, top "connect aids"
- *Search and connect the device name of “G.SoundBuds-L” or “G.SoundBuds-R”.
6. You are now ready to go.

If you need to re-pair your phone device

1. Start by removing the old pairing by going to the Settings menu.
2. Tap Accessibility.
3. Select Hearing Devices and tap Forget this device.
4. Turn your phone device off and on again and re-pair the hearing aids to the device. If you need more information, contact your hearing care professional.

【HOME SCREEN FUNCTIONS】

Main Screen:

1.Top menu bar

Use the top carousel,swipe to the mode.

2.Mode cards

3.Volume Adjustment


4.Connection status overview

5.Battery status

6.Navigational menu

7.The “Me” menu.

"Help" document could be found and request of help from your hearing are professional.

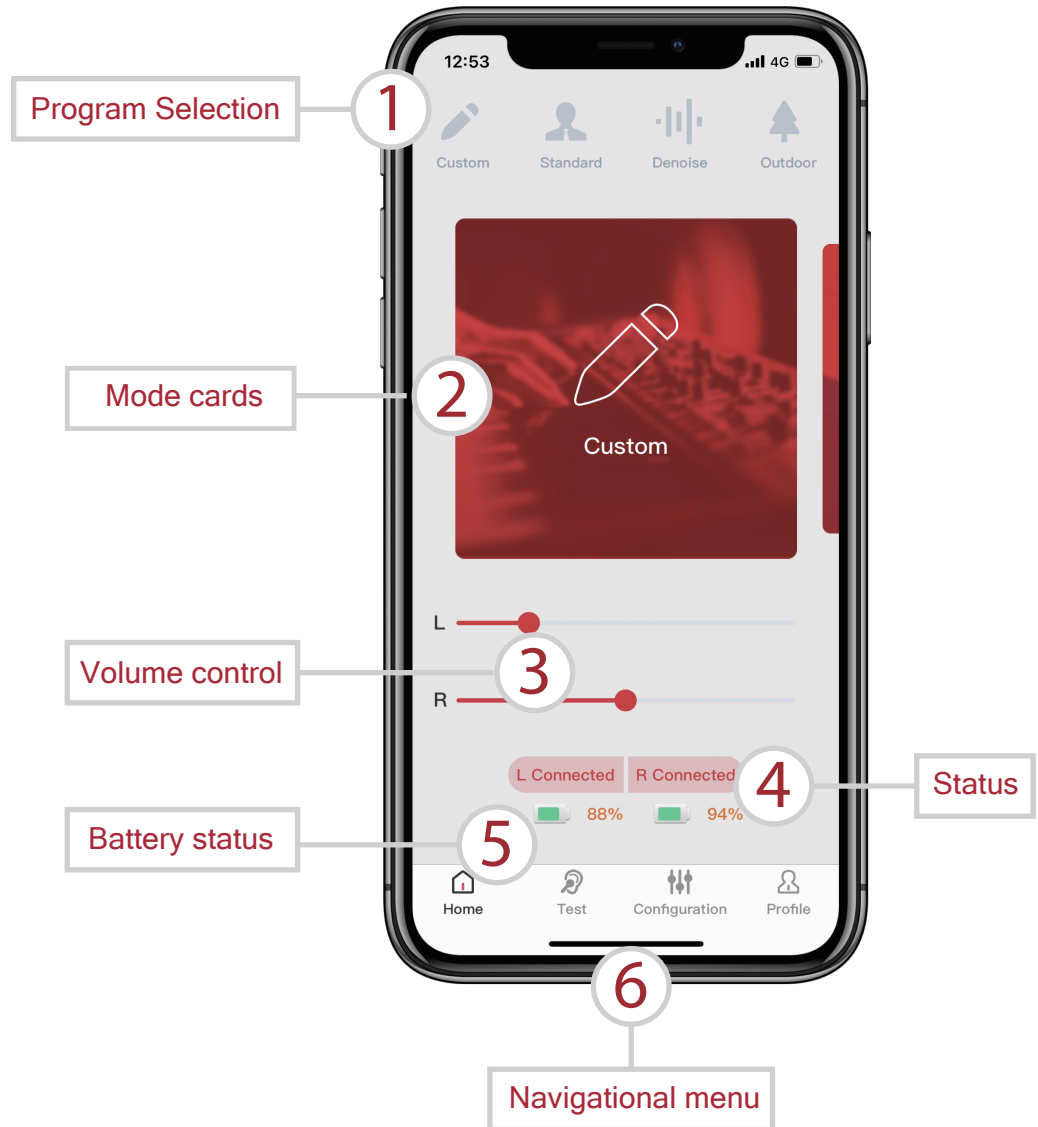

**Feature availability depends on the hearing aid model and the fitting by your hearing care professional.*

***Available if the Enhancement feature has been enabled by your hearing care professional*

1.Top menu bar:

Mode switch accordingly to best suit you.

2.Mode cards

Swipe to select a mode that suit your listening environment.

3.Adjust volume

Use the volume slider to adjust the volume of your hearing aids. If you are using the Standard program, an additional slider for the streaming device will appear, you can adjust the volume for each hearing aid separately.

4.Connection status

- Connection overview
- Dis-connected

Gray colour will be displayed if dis-connected. To re-connect, bring the hearing aid close to your device and tap the icon.

5.Battery status

Keep track of the battery status seen the progress of the batteries charging when the hearing aids are placed in the charger charging progress will be display when on charge.

6.Navigational menu

7.The “Me” menu.

Here you can learn what the app can do and contact your hearing care professional* to request help.